# Superfiller

## Superfiller

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**GB** INSTRUCTIONS FOR USE

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**NL** GEBRUIKSAANWIZING

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**GB** Attention, see instructions for use

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**FR** B. Braun Medical  
F-92107 Boulogne Cedex

**GB** B. Braun Medical Ltd.  
GB-Sheffield, S35 2PW

**DE** B. Braun Melsungen AG  
OPM  
D-34212 Melsungen

**IT** B. Braun Milano S.p.A.  
I-20161 Milano

**ES** B. Braun Medical SA  
E-08191 Rubí (Barcelona)

**NL** B. Braun Medical B.V.  
NL-5342 CW Oss

**HU** B. Braun Medical Kft.  
H-1037 Budapest

**CZ** B. Braun Medical Praha s.r.o.  
CZ-14800 Praha 4

**PL** Pofam-Poznan Sp. Z o. o.  
PL-60-573 Poznan

**SK** B. Braun Medical s.r.o.  
SK-85101 Bratislava

**AT** B. Braun Austria GmbH  
A-2344 Maria Enzersdorf

**BE** B. Braun Medical N.V./S.A.  
B-1831 Diegem

**MY** B. Braun Medical Industries  
Sdn Bhd  
MY-10810 Penang

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Protective skin barrier paste

### Product description :

Superfiller is a convenient and effective skin barrier paste, for use with colostomies, ileostomies, fistulas and urostomies. It can also be used to fill in spaces caused by skin folds, lesions or other irregularities of the skin.

### Precautions :

For external use only. Do not apply to cuts or open wounds. Contains alcohol. May sting slightly when applied. Should skin irritation occur, contact your doctor or another health professional.

### Method of Application :

Superfiller paste can be applied with a moistened finger around the stoma, filling in the spaces caused by skin folds, lesions or other irregularities of the skin. After having cut the base plate to the diameter of the stoma, place the base plate directly on the paste that has been applied to the skin. Press the wafer down on the skin around the stoma in such a way that the paste is distributed under the wafer.

### Method of Removal :

The paste is easily removed 48 hours after application. If removing the paste before this time, use warm water to remove it. It is not necessary to remove all the paste from the skin prior to a further application. The residue paste may be left in order to avoid possible irritation and the new paste applied over this residue. After use, close the tube securely.

Store in a cool and dry place , away from direct sunlight .

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