
# Flow Collector





---

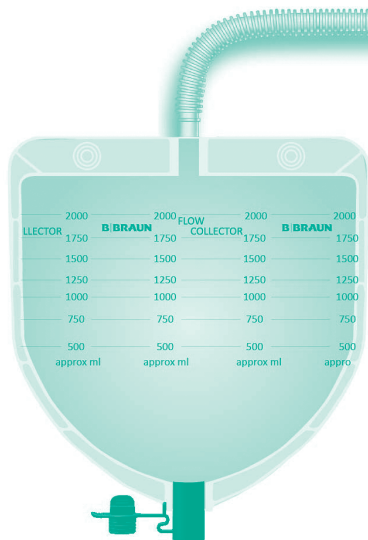- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- CS** Návod k použití
- DA** Brugsvejledning
- FI** Käyttöohje
- HU** Használati utasítás
- JA** 使用方法
- PT** Instruções de utilização
- RU** Инструкция по применению
- SK** Návod na použitie
- SV** Bruksanvisning

---

**B | BRAUN**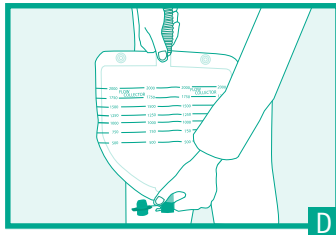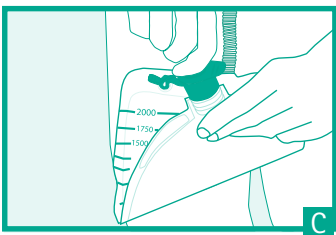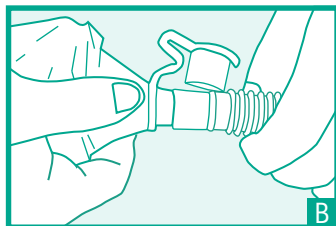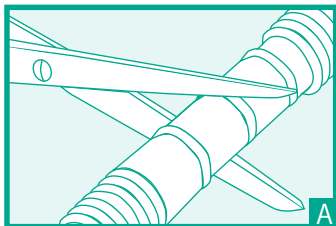

## EN Instructions for use

---

Drainable collecting system for high flow liquid stools. This product ensures the collection of high flow liquid stools being connected to the stoma bag of the patient. It is to be used with the Almarys® Twin+ pouches for High Flow, Flexima®/Softima® pouches for High Flow, Flexima®/Softima® Key Maxi and Flexima®/Softima® 3S for High Flow. The collector can be placed on the floor or higher than the floor. It must always be lower than the stoma bag.

### 1. Fitting your appliance

- Adapt the length of the tubing to the desired size with scissors by cutting it in the middle – against the edge – of the smooth areas (cutting is possible every 30 cm) (A).
- Close the green drainage cap by pressing it before the connection.
- Connect the tubing to the drainage system of the stoma bag by placing the outlet of the pouch inside the Flow Collector's tube (B).
- Disconnection after use: pull on the tubing while pressing it slightly and close the drainage system of the stoma pouch. One Flow Collector can be used for a duration of about 72 hours.

**Caution:** do not use any gelifying product in the stoma bag in order to avoid blockage in the tubing.

N.B. :

1. It is possible to get a rough measurement of the effluent volume thanks to the measuring scales in

the front of the collecting pouch. For a more precise evaluation, use a measuring collector.

2. It is possible to use B. Braun Ileo Gel+ by introducing 2 to 4 sachets in the Flow Collector (through the green cap at the lower end of the pouch). It solidifies stools: the pouch is then more silent and the emptying is easier.

### 2. Use of the drainage system

For an easier emptying, lay it on the toilets seat (C). Open the cap using the grip and taking good care to maintain it higher than the pouch.

Maintain the cap between the thumb and the index finger during the emptying (D).

Once the collecting pouch is empty, clean the cap with tap water or with a wipe (caution: do not use fluffy material) to maintain leakproofness.

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B. Braun Medical Oy
Co., Ltd.
B. Braun Medical AB